



23910 N. 19th Avenue
Building 4 - Suite 66
Phoenix, Arizona 85085 USA
(800) 243-0820 in USA
(623) 580-0556 Office
(623) 580-0115 Fax

Cleaning and Disinfection of your RadScan Slicker Products

RadScan Slickers products are made from a very safe and durable vinyl that will not be damaged by any cleaners or disinfectants commercially available for use in hospitals or clinics. However, any cleaner or disinfectant containing **acetone should not be used to clean the Slicker.**

One of the most common, effective, and low-cost disinfectants that is recommended by the CDC is hypochlorite solution (i.e. household bleach) with a solution strength between 5.25% and 6.25% diluted with water at a ratio between 10 parts water to 1 part bleach for heavy cleaning, up to 100 parts water to 1 part bleach for light wipe downs.