
## Catheter Bag Hanger Accessory

---

41" Catheter Bag Hanger with 17" of loop fastener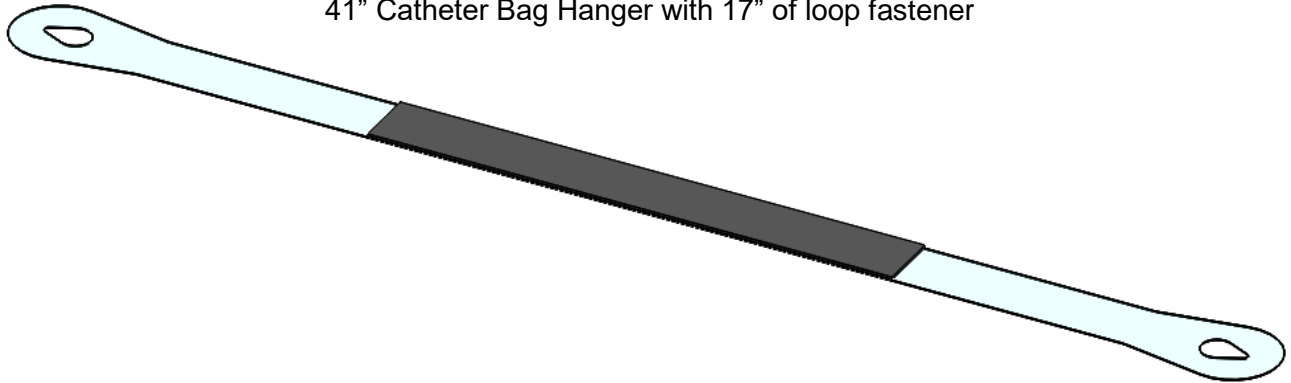

### Installation Instructions

1. Remove the pad from the CT table, and thoroughly clean and disinfect the table cradle.
2. Lay catheter bag hanger across the cradle near the foot end, perpendicular to the cradle, attaching the loop fastener to opposing fastener.
  - **IMPORTANT: Catheter bag hangers are to hold catheter bags only.**

### Tips for Use

The catheter bag hanger accessory is not shipped sterile. Clean it prior to each use with a water/bleach solution as recommended by the Center for Disease Control or other controlling authority.

## Supplementary Information

- A one-year product replacement cycle is recommended for products under normal usage
- This product meets ISO 10993 Skin Sensitivity standards
- This product is RoHS/REACH compliant
- This product is made of recyclable material and is latex free

### LIMITED 90 DAY WARRANTY

RadScan Medical Equipment, Inc. warrants this medical table product as follows:

**What is covered:** Defects in material or workmanship  
**For How Long:** Ninety days from receipt.  
**What We Will Do:** Replace the product.

What You Must Do to Get a Replacement Product:

1. Contact your vendor or email, FAX, or call us.
2. Give your name, address, and date of purchase.
3. We will arrange disposition of the defective product and provide a replacement.

**Other Limitations:** This warranty does not cover products that have been lost, stolen, or damaged by accident, misuse, or unauthorized modifications.

**No Other Warranty:** We make no other warranty, express or implied, to you or any other person or entity.

**No Liability for Consequential Damages:** We will not be liable for special, incidental, consequential, indirect or similar damages. This means we are not responsible or liable for damages or costs incurred due to loss of time, loss of savings, loss of profits or revenue, or loss of use of the product. Additionally, we are not responsible or liable for damages or costs in connection with claims by others, inconvenience, or similar costs. In no event will our liability for any damages to you, any other person or entity, exceed the replacement cost of the product.

**No Unauthorized Changes of Warranty:** Statements by agents, employees, and distributors such as advertising or presentations, do not constitute warranties by RadScan Medical Equipment, do not bind RadScan Medical Equipment, and should not be relied upon.

**Country/State Law Rights:** Some country/states do not allow the exclusion or limitation of incidental or consequential damages, so some of the above limitations may not apply to you.

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RadScan Medical Equipment, Inc. garantiza este producto de tabla médica de la siguiente manera: