
GE HiSpeed Advantage CT Skirt™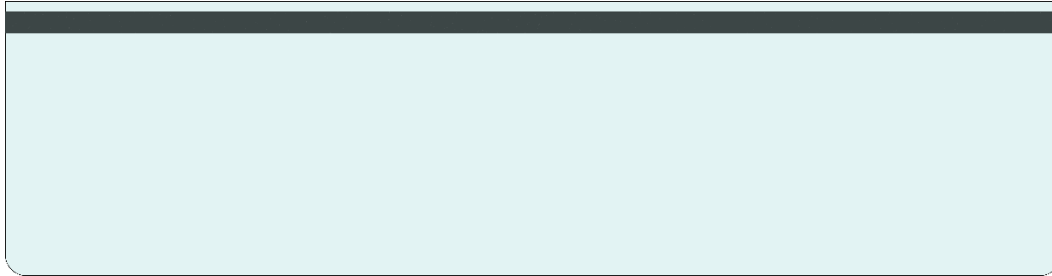

English

Device Intended Purpose

Protective vinyl cover for medical imaging equipment intended to prevent the accidental ingress to the machine of human bodily fluids or pharmaceutical liquids, which could interfere with working mechanisms and cause electromechanical failure. (Device should only be used by medical imaging professionals for patients prescribed for CT, PET, or Nuclear/Molecular imaging.)

Installation Instructions

1. The table cradle rests between two long, thin table cover panels. Each cover panel has a long indentation ending in a control panel located at the head end of the table. The control panel is in the deepest section of the indentation. The skirts will be mounted in the shallow portion of the indentation.
2. Thoroughly clean and dry the table side panels, paying particular attention to the indentation on each side.
3. Open the skirts on a large, flat, clean surface (*see Tips for Use below*).
4. Check that the strips of self-adhesive hook fastener are straight and fully attached to the loop fastener strips that are welded to the two skirts.
5. Peel and remove the backing from the fastener on one skirt, exposing the adhesive.
 - Position and attach the skirt so the adhesive fastener will mount in the shallow indentation on the side of the table as shown in the figure below.
 - Once positioned, press down firmly along the length of the fastener strip to set the adhesive. The skirt should drape down evenly.
 - Repeat the process with the second skirt on the opposite side of the table.

Tips for Use

The skirts and other accessories are not shipped sterile. Clean the skirts prior to each use with a water/bleach solution as recommended by the Center for Disease Control or other controlling authority.

Supplementary Information

- A one-year product replacement cycle is recommended for products under normal usage
- This product meets ISO 10993 Skin Sensitivity standards
- This product is RoHS/REACH compliant
- This product is made of recyclable material and is latex free

LIMITED 90 DAY WARRANTY

RadScan Medical Equipment, Inc. warrants this medical table product as follows:

What is covered: Defects in material or workmanship
For How Long: Ninety days from receipt.
What We Will Do: Replace the product.

What You Must Do to Get a Replacement Product:

1. Contact your vendor or email, FAX, or call us.
2. Give your name, address, and date of purchase.
3. We will arrange disposition of the defective product and provide a replacement.

Other Limitations: This warranty does not cover products that have been lost, stolen, or damaged by accident, misuse, or unauthorized modifications.

No Other Warranty: We make no other warranty, express or implied, to you or any other person or entity.

No Liability for Consequential Damages: We will not be liable for special, incidental, consequential, indirect or similar damages. This means we are not responsible or liable for damages or costs incurred due to loss of time, loss of savings, loss of profits or revenue, or loss of use of the product. Additionally, we are not responsible or liable for damages or costs in connection with claims by others, inconvenience, or similar costs. In no event will our liability for any damages to you, any other person or entity, exceed the replacement cost of the product.

No Unauthorized Changes of Warranty: Statements by agents, employees, and distributors such as advertising or presentations, do not constitute warranties by RadScan Medical Equipment, do not bind RadScan Medical Equipment, and should not be relied upon.

Country/State Law Rights: Some country/states do not allow the exclusion or limitation of incidental or consequential damages, so some of the above limitations may not apply to you.

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RadScan Medical Equipment, Inc. garantiza este producto de tabla médica de la siguiente manera: