
E8016AS and E8016AY

GE Omega-III, Omega-IV Angio Slicker®
Protective Medical Table Cover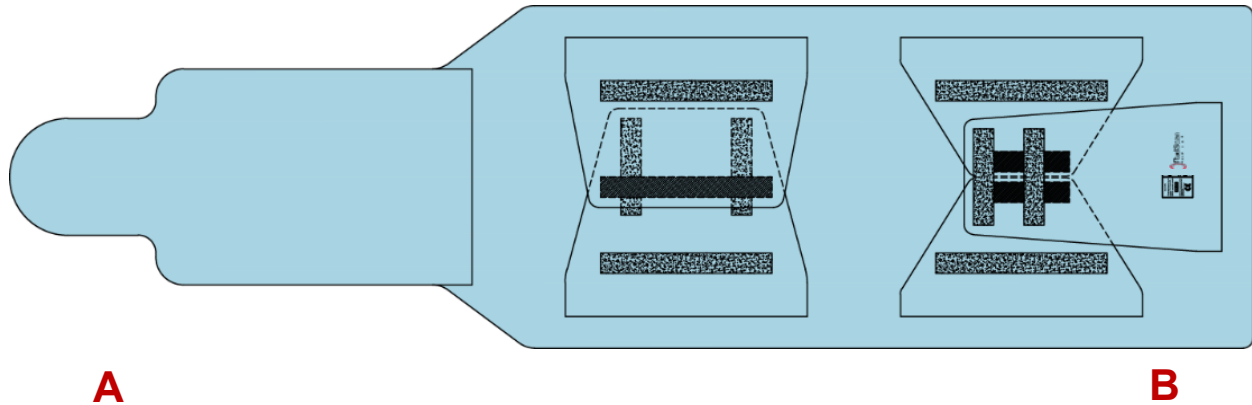

A = Head End
B = Foot End

Device Intended Purpose

Protective vinyl cover for medical imaging equipment intended to prevent the accidental ingress to the machine of human bodily fluids or pharmaceutical liquids, which could interfere with working mechanisms and cause electromechanical failure. (Device should only be used by medical imaging professionals for patients prescribed for CT, PET, or Nuclear/Molecular imaging.)

Installation Instructions

1. Remove the pad from the Omega table, and thoroughly clean and disinfect the table cradle.
2. Install the pad into the Angio Slicker by first inserting the head end of the pad into the head end of the Slicker.
3. Close the center and foot end flaps of the Slicker over the pad, connecting hook and loop fasteners together.
4. Self-adhesive hook fastener is provided to attach to the table. Remove backing on fastener and adhere to table.

Tips for Use

- The Angio Slicker and any accessories are not shipped sterile. Clean the Slicker and any accessories prior to each use with a water/bleach solution as recommended by the Center for Disease Control or other controlling authority.
- A sheet or blanket is suggested during use for the following reasons:
 - o To aid in cleanup and table sanitization
 - o To aid in sliding and positioning a patient
 - o To mitigate coldness in air conditioned rooms for increased patient comfort
- The Slicker is designed to fit the specified original table and pad provided by your machine manufacturer.
 - o Using the Slicker on a table with a different configuration may decrease the contaminant protection provided by the Slicker.
 - o The Slicker should remain on the table at all times to protect against unpredictable situations such as vomiting or patient incontinence.

Supplementary Information

- A one-year product replacement cycle is recommended for products under normal usage
- This product meets ISO 10993 Skin Sensitivity standards
- This product is RoHS/REACH compliant
- This product is made of recyclable material and is latex free

LIMITED 90 DAY WARRANTY

RadScan Medical Equipment, Inc. warrants this medical table product as follows:

What is covered: Defects in material or workmanship
For How Long: Ninety days from receipt.
What We Will Do: Replace the product.

What You Must Do to Get a Replacement Product:

1. Contact your vendor or email, FAX, or call us.
2. Give your name, address, and date of purchase.
3. We will arrange disposition of the defective product and provide a replacement.

Other Limitations: This warranty does not cover products that have been lost, stolen, or damaged by accident, misuse, or unauthorized modifications.

No Other Warranty: We make no other warranty, express or implied, to you or any other person or entity.

No Liability for Consequential Damages: We will not be liable for special, incidental, consequential, indirect or similar damages. This means we are not responsible or liable for damages or costs incurred due to loss of time, loss of savings, loss of profits or revenue, or loss of use of the product. Additionally, we are not responsible or liable for damages or costs in connection with claims by others, inconvenience, or similar costs. In no event will our liability for any damages to you, any other person or entity, exceed the replacement cost of the product.

No Unauthorized Changes of Warranty: Statements by agents, employees, and distributors such as advertising or presentations, do not constitute warranties by RadScan Medical Equipment, do not bind RadScan Medical Equipment, and should not be relied upon.

Country/State Law Rights: Some country/states do not allow the exclusion or limitation of incidental or consequential damages, so some of the above limitations may not apply to you.

For reorder in USA:
Please contact GE Medical Systems Diagnostic Imaging Accessories at: (800) 437-1171

Notice to the user and/or patient: If a serious incident occurs in relation to the device, please contact RadScan Medical Equipment, Inc. and the competent authority of the Member State in which the user and/or patient is established.

RadScan Medical Equipment, Inc.
23910 N 19th Ave, Bldg 4, Ste 66
Phoenix, AZ 85085 USA
www.radscanmedical.com

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RadScan Medical Equipment, Inc. garantiza este producto de tabla médica de la siguiente manera:
MDSS GmbH Schiffgraben 41