


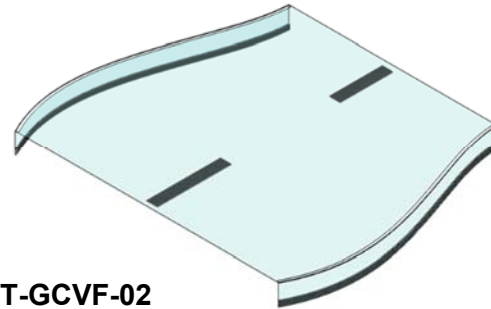
# CT-GCVF-02 and CT-HSFS-01

## Footswitch Slicker<sup>®</sup> Protective Cover

Protective Medical Footswitch Cover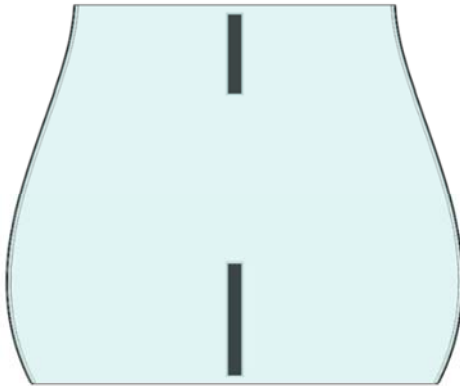

**CT-GCVF-02**  
for GE Discovery CT750 HD & VCT Systems

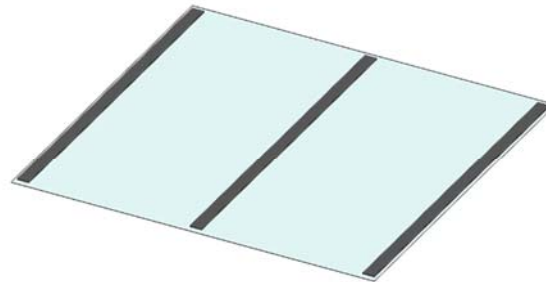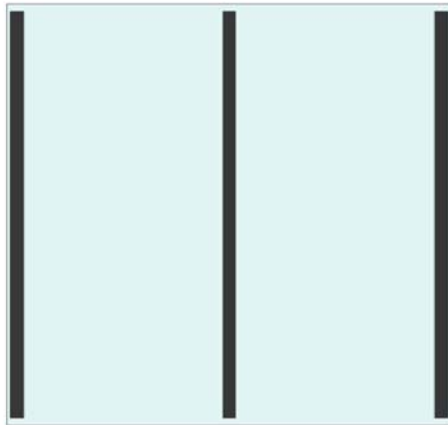**CT-HSFS-01**  
for GE HiSpeed Advantage/Pet Advanced Systems

### Device Intended Purpose

Protective vinyl cover for medical imaging equipment intended to prevent the accidental ingress to the machine of human bodily fluids or pharmaceutical liquids, which could interfere with working mechanisms and cause electromechanical failure. (Device should only be used by medical imaging professionals for patients prescribed for CT, PET, or Nuclear/Molecular imaging.)

### Installation Instructions

1. Thoroughly clean and dry the footswitch base.
2. Open the cover on a large, flat, clean surface (*see Tips for Use below*).
3. Detach the two side strips of self-adhesive loop (fuzzy) fastener from the hook (sharp) fastener strips of the cover (do not detach the center strips).
4. Remove the backings from each loop fastener strip and apply one to each side of the footswitch base.
  - Make sure to position and attach the fastener strips below the foot pedals. Once attached, press them down firmly across their entire length to set the adhesive.
5. Lay the cover on the footswitch housing—fastener-side down, and attach each side of the cover to the loop fastener just placed at each side of the base.
  - Once the sides are attached, make effort to position the cover so that the center fastener strips align down the center of the footswitch housing.
  - Detach one side of the cover from the footswitch housing base and lay it back, holding the center fastener strips down in place so as not to lose their placement.
  - Remove the backings of the center self-adhesive loop fastener strips (again, do not detach the actual strips) and mount to the footswitch housing, pressing the center strips down firmly to set the adhesive.
  - Fold the rest of the cover back over to reattach the side at the footswitch base.

## Tips for Use

- The Footswitch Slicker and other accessories are not shipped sterile. Clean the Footswitch Slicker prior to each use with a water/bleach solution as recommended by the Center for Disease Control or other controlling authority.

## Supplementary Information

- A one-year product replacement cycle is recommended for products under normal usage
- This product meets ISO 10993 Skin Sensitivity standards
- This product is RoHS/REACH compliant
- This product is made of recyclable material and is latex free

## LIMITED 90 DAY WARRANTY

RadScan Medical Equipment, Inc. warrants this medical table product as follows:

**What is covered:** Defects in material or workmanship  
**For How Long:** Ninety days from receipt.  
**What We Will Do:** Replace the product.

What You Must Do to Get a Replacement Product:

1. Contact your vendor or email, FAX, or call us.
2. Give your name, address, and date of purchase.
3. We will arrange disposition of the defective product and provide a replacement.

**Other Limitations:** This warranty does not cover products that have been lost, stolen, or damaged by accident, misuse, or unauthorized modifications.

**No Other Warranty:** We make no other warranty, express or implied, to you or any other person or entity.

**No Liability for Consequential Damages:** We will not be liable for special, incidental, consequential, indirect or similar damages. This means we are not responsible or liable for damages or costs incurred due to loss of time, loss of savings, loss of profits or revenue, or loss of use of the product. Additionally, we are not responsible or liable for damages or costs in connection with claims by others, inconvenience, or similar costs. In no event will our liability for any damages to you, any other person or entity, exceed the replacement cost of the product.

**No Unauthorized Changes of Warranty:** Statements by agents, employees, and distributors such as advertising or presentations, do not constitute warranties by RadScan Medical Equipment, do not bind RadScan Medical Equipment, and should not be relied upon.

**Country/State Law Rights:** Some country/states do not allow the exclusion or limitation of incidental or consequential damages, so some of the above limitations may not apply to you.

**Notice to the user and/or patient:** If a serious incident occurs in relation to the device, please contact RadScan Medical Equipment, Inc. and the competent authority of the Member State in which the user and/or patient is established.

RadScan Medical Equipment, Inc.  
23910 N 19th Ave, Bldg 4, Ste 66  
Phoenix, AZ 85085 USA  
[www.radscanmedical.com](http://www.radscanmedical.com)

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RadScan Medical Equipment, Inc. garantiza este producto de tabla médica de la siguiente manera:
MDSS GmbH Schiffgraben 41