
CT Slicker[®] Cushion

Protective Medical Table Cushion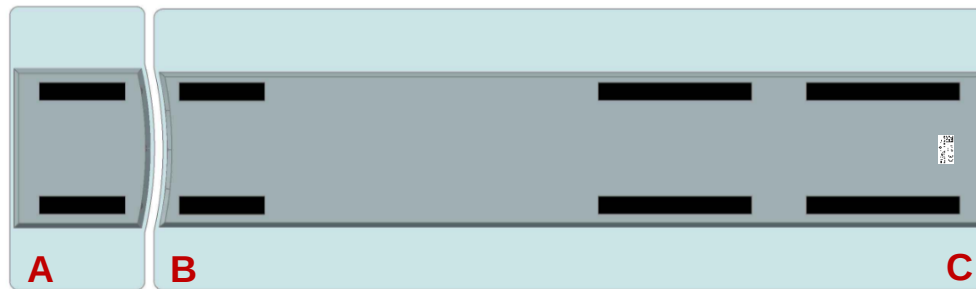

*** Images for reference only, your product may vary in size and shape ***

- A** = Extender
- B** = Head End
- C** = Foot End

Device Intended Purpose

Protective vinyl cover for medical imaging equipment intended to prevent the accidental ingress to the machine of human bodily fluids or pharmaceutical liquids, which could interfere with working mechanisms and cause electromechanical failure. (Device should only be used by medical imaging professionals for patients prescribed for CT, PET, or Nuclear/Molecular imaging.)

Installation Instructions

1. Remove the pad from the CT table, and thoroughly clean and disinfect the table cradle.
2. Affix the new CT Slicker Cushion onto the CT table by aligning the fastener on the Slicker Cushion to the opposing fastener on the table.
 - Certain product versions will have opposing, self-adhesive fastener attached to the fastener on the Slicker Cushion. This self-adhesive fastener is to be adhered onto the table by the user to secure the Slicker Cushion in place.
3. If an extender cushion is included, repeat steps 1 and 2 for it as well.
4. If a skirt set is included, mount the skirts to the non-moving table rails—one on each side of the table, draping them over the accordion base.
 - First attach provided hook fastener $\frac{1}{2}$ " below the bottom of the lip near the top of the table rails.

Tips for Use

- The CT Slicker Cushion and accessories are not shipped sterile. Clean the Slicker Cushion and any accessories prior to each use with a water/bleach solution as recommended by the Center for Disease Control or other controlling authority.
- The Slicker Cushion is designed to be used in conjunction with the restraint belts on your table; allowing the Slicker Cushion flaps to drape during scans may cause edge artifacting on some machines.
- A sheet or blanket is suggested during use for the following reasons:
 - To aid in cleanup and table sanitization
 - To aid in sliding and positioning a patient
 - To mitigate coldness in air conditioned rooms for increased patient comfort
- The Slicker Cushion is designed to fit the specified original table provided by your machine manufacturer.
 - Using the Slicker Cushion on a table with a different configuration may decrease the contaminant protection provided by the cushion.
 - The Slicker Cushion should remain on the table at all times to protect against unpredictable situations such as vomiting or patient incontinence.

Supplementary Information

- A one-year product replacement cycle is recommended for products under normal usage
- This product meets ISO 10993 Skin Sensitivity standards
- This product is RoHS/REACH compliant
- This product is made of recyclable material and is latex free

LIMITED 90 DAY WARRANTY

RadScan Medical Equipment, Inc. warrants this medical table product as follows:

What is covered: Defects in material or workmanship
For How Long: Ninety days from receipt.
What We Will Do: Replace the product.

What You Must Do to Get a Replacement Product:

1. Contact your vendor or email, FAX, or call us.
2. Give your name, address, and date of purchase.
3. We will arrange disposition of the defective product and provide a replacement.

Other Limitations: This warranty does not cover products that have been lost, stolen, or damaged by accident, misuse, or unauthorized modifications.

No Other Warranty: We make no other warranty, express or implied, to you or any other person or entity.

No Liability for Consequential Damages: We will not be liable for special, incidental, consequential, indirect or similar damages. This means we are not responsible or liable for damages or costs incurred due to loss of time, loss of savings, loss of profits or revenue, or loss of use of the product. Additionally, we are not responsible or liable for damages or costs in connection with claims by others, inconvenience, or similar costs. In no event will our liability for any damages to you, any other person or entity, exceed the replacement cost of the product.

No Unauthorized Changes of Warranty: Statements by agents, employees, and distributors such as advertising or presentations, do not constitute warranties by RadScan Medical Equipment, do not bind RadScan Medical Equipment, and should not be relied upon.

Country/State Law Rights: Some country/states do not allow the exclusion or limitation of incidental or consequential damages, so some of the above limitations may not apply to you.

Notice to the user and/or patient: If a serious incident occurs in relation to the device, please contact RadScan Medical Equipment, Inc. and the competent authority of the Member State in which the user and/or patient is established.

RadScan Medical Equipment, Inc.
23910 N 19th Ave, Bldg 4, Ste 66
Phoenix, AZ 85085 USA
www.radscanmedical.com

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RadScan Medical Equipment, Inc. garantiza este producto de tabla médica de la siguiente manera:
MDSS GmbH Schiffgraben 41